SAMPLE HANDLING

For DNA testing at the University of Missouri JLLP valid for Rottweilers and Black Russian Terriers

<u>Blood Sample</u> - The ideal sample for DNA extraction is 3 to 7cc's of whole blood, in **purple-topped (EDTA)** tubes (one or several, depending on tube size). For very small dogs, 1 to 2 ccs should be sufficient. Please do not send less than 1 cc. The blood sample needs only to be put in the tubes and rocked gently a few times to distribute the anticoagulant - do **not** spin, extract serum, or do anything further. Refrigerate if the sample is being held for any time before shipping, but do not hold the sample longer than 1 week before shipping or it may become unusable.

<u>Frozen Semen</u> - Frozen semen stored from deceased sires or affected dogs can be a source of DNA for testing. Please send about 1 breeding unit. The straws or pellets do not need to be shipped frozen, but do pack them in a crush-proof & leak-proof container. Special handling fee is \$40 for this sample, in addition to the regular testing fee.

<u>Tissue Sample</u> - DNA can be extracted from any cell-rich tissue. If a dog is to be tested post-mortem, a 1" cube (or equivalent) of tongue, other muscle, spleen, kidney, or liver will provide a large amount of DNA (one tissue is sufficient – do not send multiple tissues). Tissue samples should be placed in a clearly labeled freezer bag or other sterile container and frozen. **DO NOT place in formalin!** Place the bagged tissue inside another bag, freeze, and ship with a frozen cool pack (do not use dry ice, or ice cubes placed in a ziplock bag). If this is the only sample (no blood sample available), add special handling fee of \$40 to regular testing fee.

Label sample with the following:

call name - owner's last name (If samples from several dogs are sent together, number samples and forms)

The *Individual Dog Information Form & Survey* that follow this instruction sheet should be completed, and a *pedigree copy*, if available, should be included with the sample...

TESTING FEE is \$65; check or money order payable to "University of Missouri", or include credit card information in payment section of form.

Shipping - Ideally, blood samples should be shipped immediately, tissues should be frozen first. If samples are held for a couple days or over a weekend, blood must be refrigerated, and tissue samples must be kept frozen. Ship for **next day** delivery (FedEx, US Mail-Express service, or UPS). **Do not send on a Friday** - there will not be anyone to accept the delivery on a weekend, and the sample could be unusable by Monday. Pack in a small insulated container (Styrofoam box, small cooler or insulated lunch bag), with one or more cool packs - it is important that blood samples be kept cool but not frozen, and tissue samples be kept as frozen as possible.

The delivery address is;

Dr. Gary Johnson - JLPP Testing 320 Connaway Hall-UMC 1500 Bouchelle Ave University of Missouri Columbia, MO 65211

(NOTE: if UPS does not recognize 320 Connaway as a valid address, use 201 Connaway)

If you need clarification, or have any questions about any of these procedures, please contact Liz Hansen by phone (573-884-3712), email (HansenL@missouri.edu), or regular mail (321 Connaway Hall, University of Missouri, Columbia, MO 65211).

UMC JLPP DNA TESTING & RESEARCH

Blood - Tissue - FTA-swab - semen - other Registered Name			Breed: Rottweiler -or- Black Russian Terrier		
					Call name
Reg# Birth Date			Male / Female Intact / Neutered		Male / Female Intact / Neutered
Microchip or Tattoo:					Color
Test E	Being Requested:	JLPP – Juvenile	Laryr	ngeal Pa	ralysis & Polyneuropathy
Owne	r: name		Vete	erinarian	
	address				
	city-st-zip				
phone (day)					
	phone (eve)				
	EMAIL			EMAIL	
PAYN	IENT INFORMATION:□	Check or money or	der p	ayable to	er Veterinarian Both o "University of Missouri" enclosed
	Cardholder nam	ne:			Exp Date:
FEE					
Does	this dog exhibit any of t	he following cond	litions	s? (Plea	se attach history for any Yes answer)
Y - N	Allergies	Υ	- N	Digestiv	ve difficulties
Y - N	Arthritis	Υ	- N	Heart P	Problems
Y - N	Autoimmune Disorders	Y	- N	Hernia	(where?)
Y - N	Bite or Tooth Abnormali	ties Y	- N	Reprod	luctive Problems
Y - N	Cancer / Tumors	Y	- N	Seizure	es :
Y - N	Cataracts / Vision Probl	ems Y	- N	Skin / C	Coat Problems
Y - N	Deafness / Hearing Imp	aired Y	- N	Skeleta	al Abnormalities (Hip Dysplasia, etc.)
Y - N	Hindlimb weakness/para	alysis Y	- N	Temper	rament Problems (shy, aggressive, etc.)
other ((please list):				
Comm	nents / Questions / Concer	ns?			
followi be rep email	ing the test may be stored ported only to the owner lis or FAX; and I have supplie	for potential future sted on this form ar	researd to t	arch; I ur he veteri	g; I understand that DNA left over nderstand that the results of this test will inarian (if requested) listed here, via nation, to the best of my knowledge.
Signe	d:				date